Quattro™ Air

Quattro™ FX

Mirage Quattro™

Mirage Liberty™

Vented full face mask

User guide

English | Deutsch | Français | Italiano | Español | Português | Nederlands

Ελληνικά | Polski | Magyar | Русский | Slovenčina | Türkçe | Český

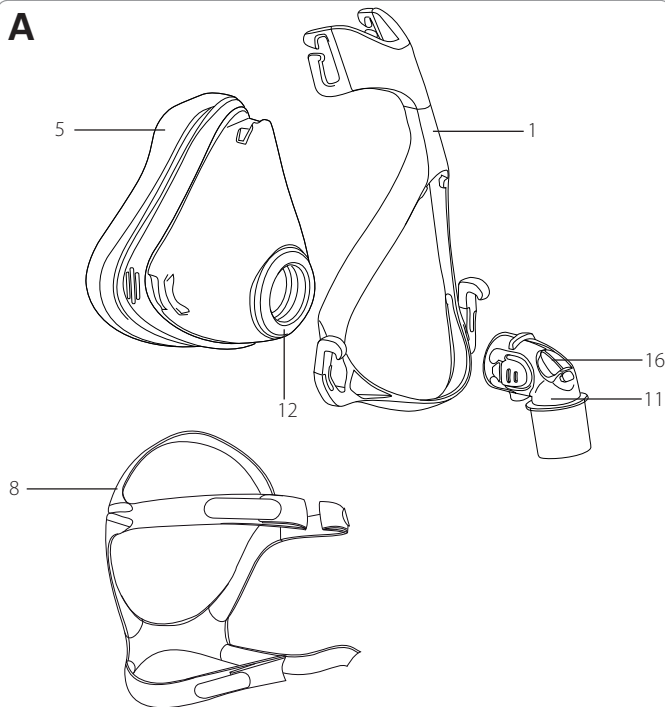
Contents

Quattro™ Air / Quattro™ Air for Her	3
Quattro™ FX / Quattro™ FX for Her	7
Mirage Quattro™	13
Mirage Liberty™	19

Quattro™ Air / Quattro™ Air for Her










A





Quattro™ Air / Quattro™ Air for Her

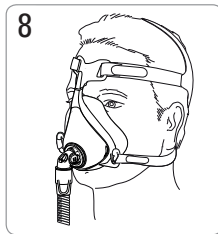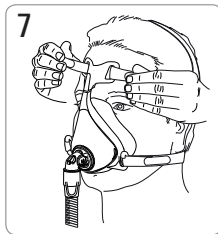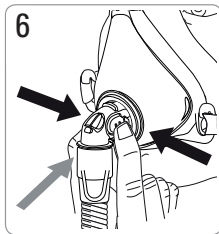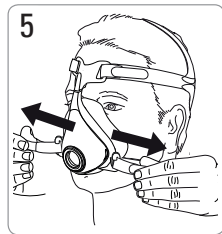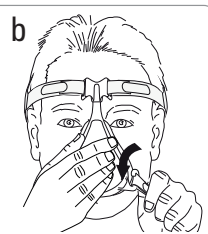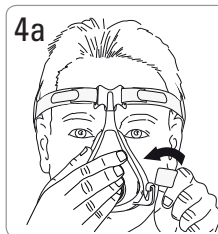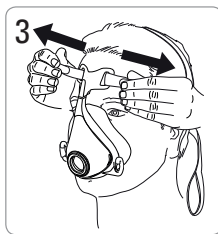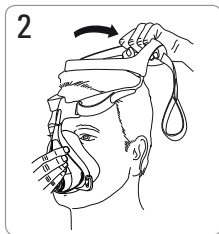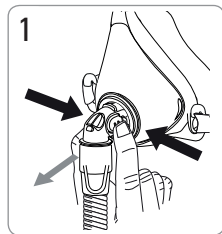Fitting / Anlegen / Mise en place / Applicazione / Colocación / Colocação / Opzetten / Τοποθέτηση της μάσκας / Zakładanie maski / A maszk felhelyezése / Установка маски / Nastavenie masky / Maskenin Takılması / Nasazení masky




Quattro™ Air / Quattro™ Air for Her

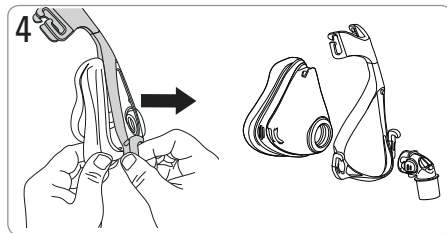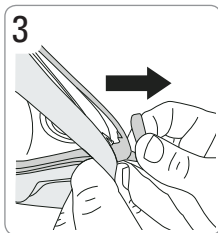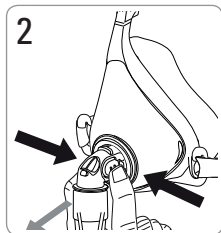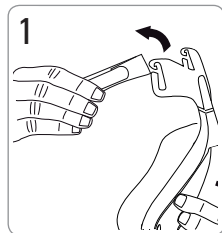Disassembly / Auseinanderbau / Démontage / Smontaggio / Desmontaje / Desmontagem / Demontage / Αποσυναρμολόγηση /
Demontaž / Szétszerelés / Разборка / Rozobratie / Sökme / Demontáž

Quattro™ Air / Quattro™ Air for Her

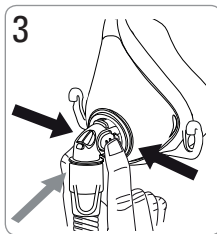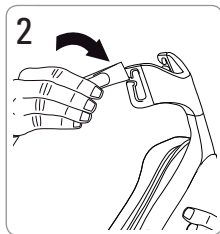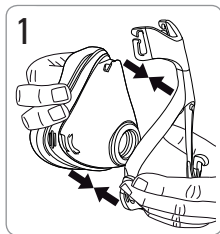
Reassembly / Zusammenbau / Remontage / Riasssemblaggio / Montaje / Montagem / Opnieuw monteren / Επανασυναρμολόγηση / Składanie / Összeszerelés / Повторная сборка / Opäťovné zloženie / Yeniden Montaj / Opěťovné sestavení

Quattro™ FX / Quattro™ FX for Her

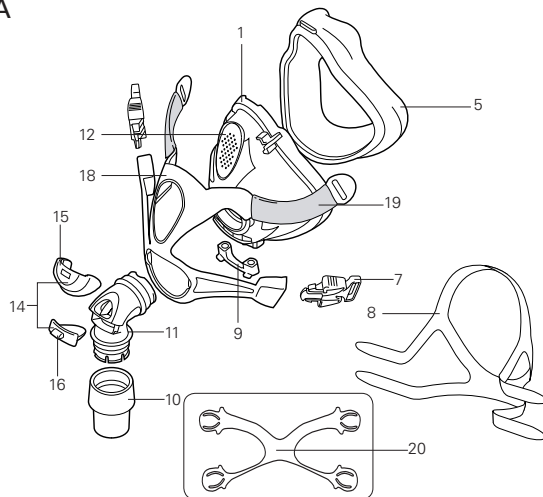A







B




Quattro™ FX / Quattro™ FX for Her

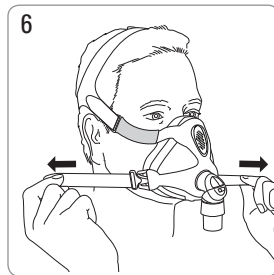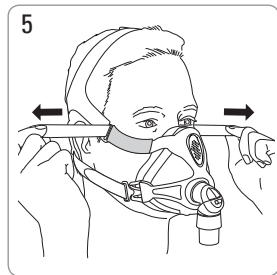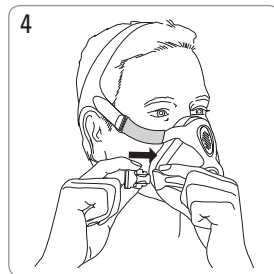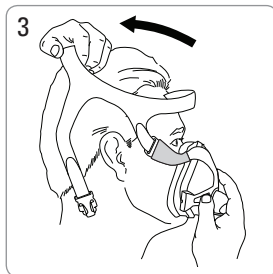
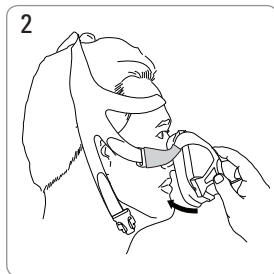
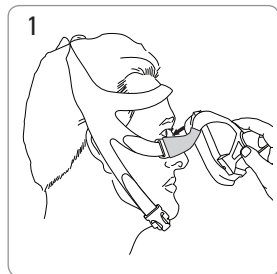
Fitting / Anlegen / Mise en place / Applicazione / Colocación / Colocação / Opzetten / Τοποθέτηση της μάσκας / Zakładanie maski / A maszk felhelyezése / Установка маски / Nastavenie masky / Maskenin Takılması / Nasazení masky

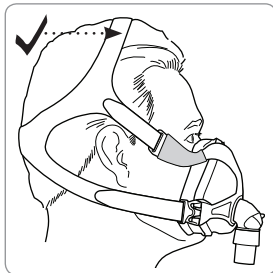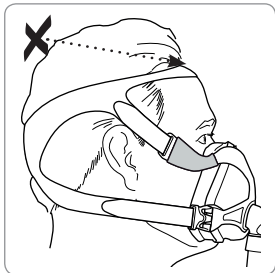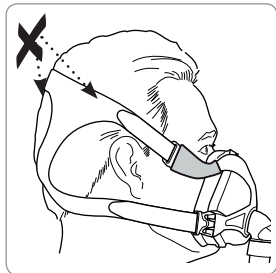Headgear check / Kontrolle des Kopfbandes / Vérification du harnais / Verifica copricapo / Verificación del arnés / Verificação do arnés / Controle hoofdband / Έλεγχος μάντων κεφαλής / Kontrola założenia części nagłownej / Fejpánt ellenőrzése / Проверка наголовника / Správne nasadenie náhlavnej súpravy / Başlık kontrolü / Kontrola náhlavní soupravy







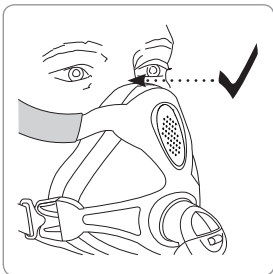
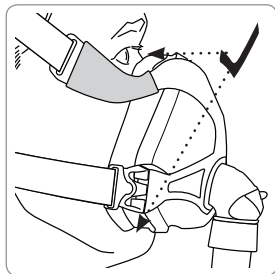
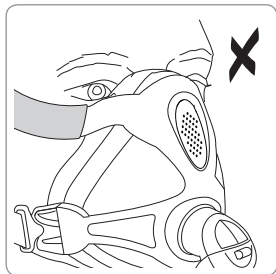
Cushion check / Kontrolle des Maskenkissens / Vérification de la bulle / Verifica cuscinetto / Verificación de la almohadilla / Verificação da almofada / Controle kussentje / Έλεγχος μαξιλαριού / Kontrola założenia uszczelki / Párna ellenőrzése / Проверка уплотнителя / Správne nasadenie podušky / Yastık kontrolü / Kontrola polštářku

Quattro™ FX / Quattro™ FX for Her

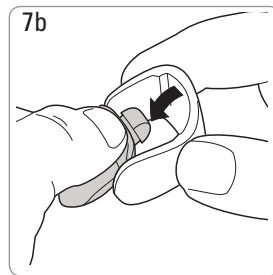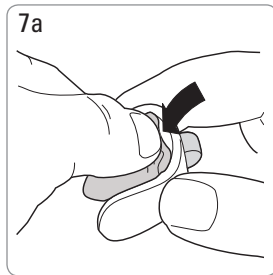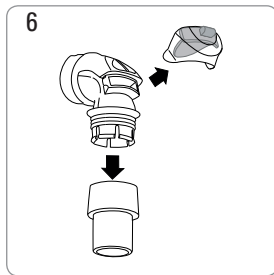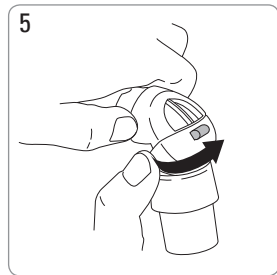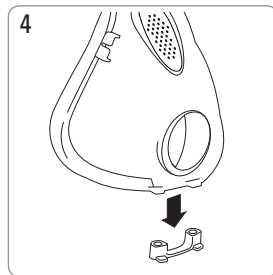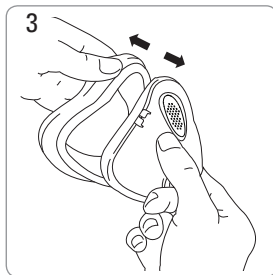
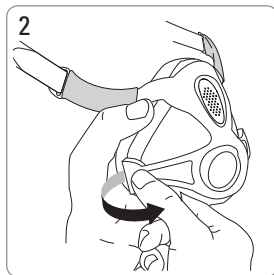
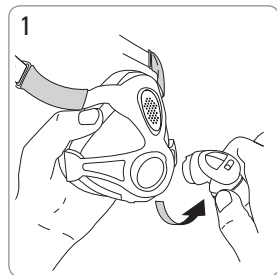
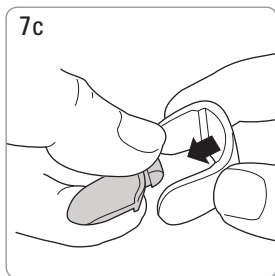
Disassembly / Auseinanderbau / Démontage / Smontaggio / Desmontaje / Desmontagem / Demontage / Αποσυναρμολόγηση /
Demontaż / Szétszerelés / Разборка / Rozobratie / Sökme / Demontáž







Quattro™ FX / Quattro™ FX for Her

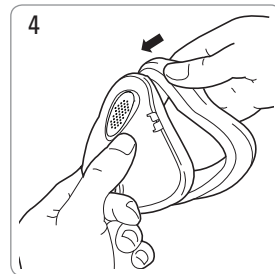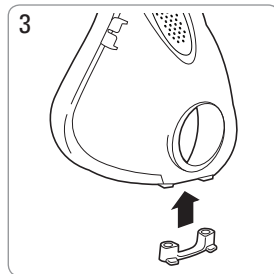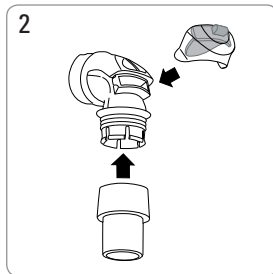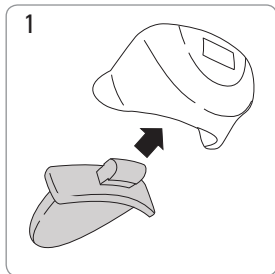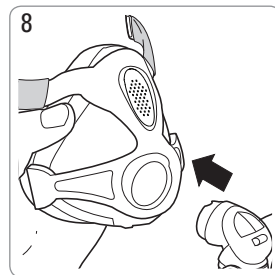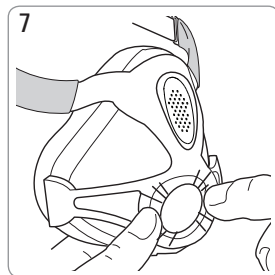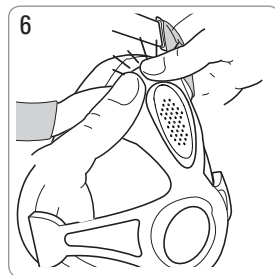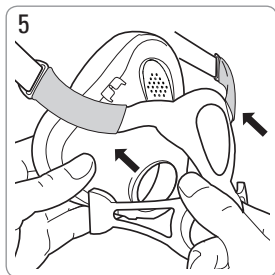Reassembly / Zusammenbau / Remontage / Riassemblaggio / Montaje / Montagem / Opnieuw monteren / Επανασυναρμολόγηση / Składanie / Összeszerelés / Повторная сборка / Opätovné zloženie / Yeniden Montaj / Opětovné sestavení






Mirage Quattro™







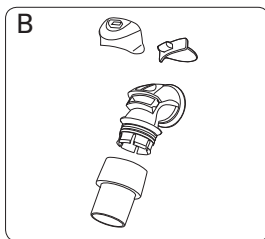
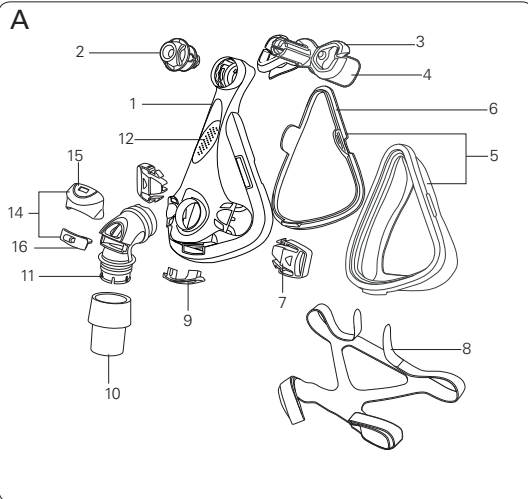
Mirage Quattro™

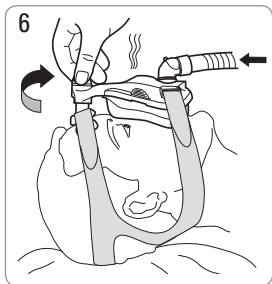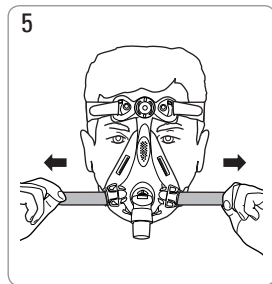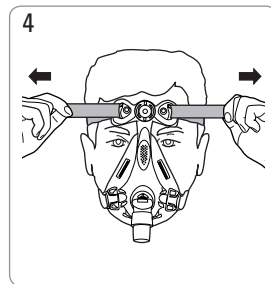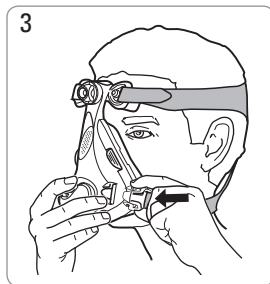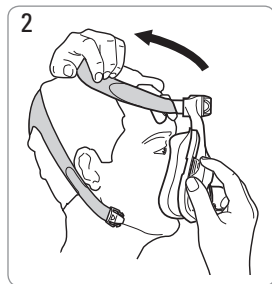Fitting / Anlegen / Mise en place / Applicazione / Colocación / Colocação / Opzetten / Τοποθέτηση της μάσκας / Zakładanie maski /
A maszk felhelyezése / Установка маски / Nastavenie masky / Maskenin Takılması / Nasazení masky


Mirage Quattro™

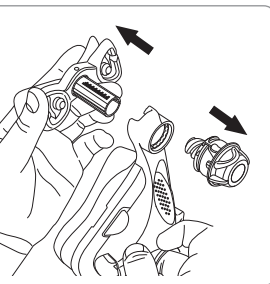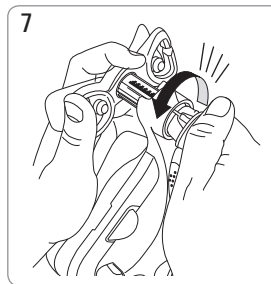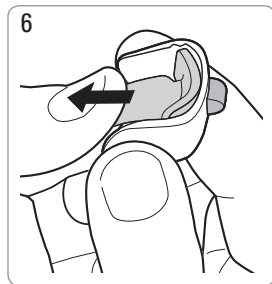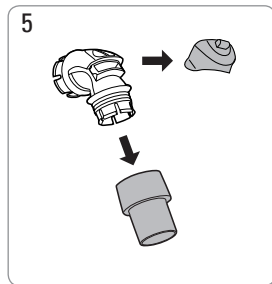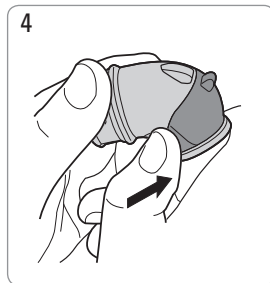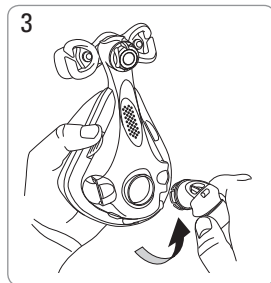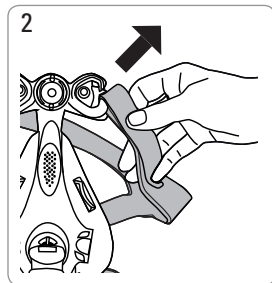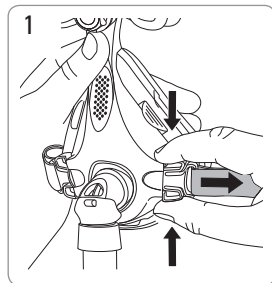
Disassembly / Auseinanderbau / Démontage / Smontaggio / Desmontaje / Desmontagem / Demontage / Αποσυναρμολόγηση /
Demontaž / Szétszerelés / Разборка / Rozobratie / Sökme / Demontáž


8

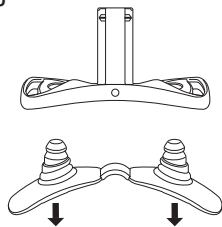
9

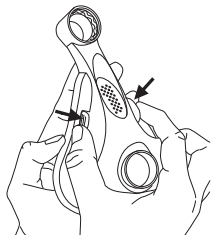
10

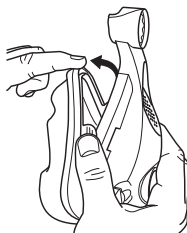11










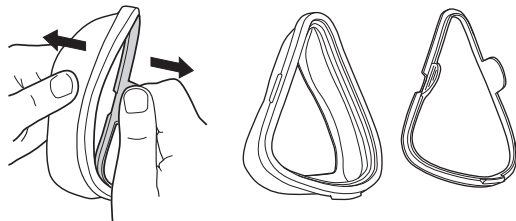
12



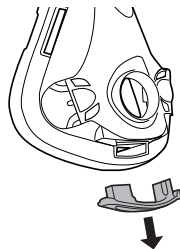
Mirage Quattro™

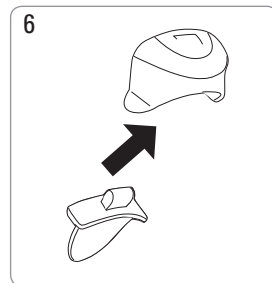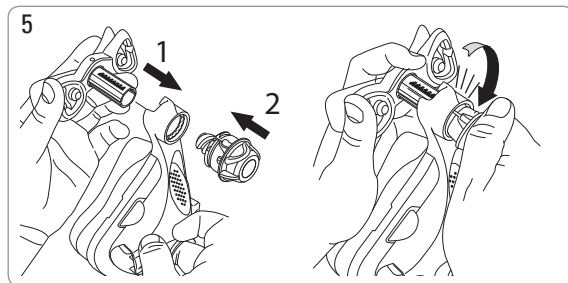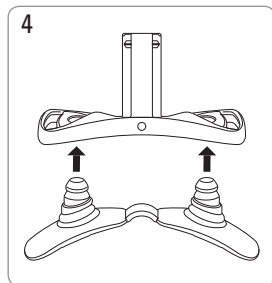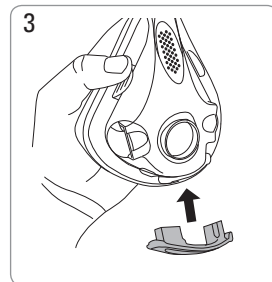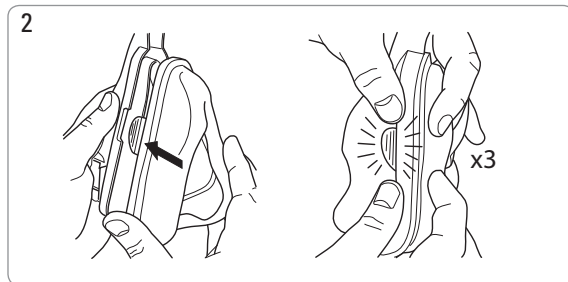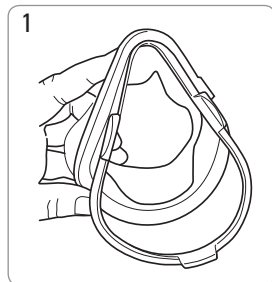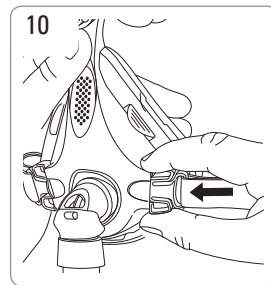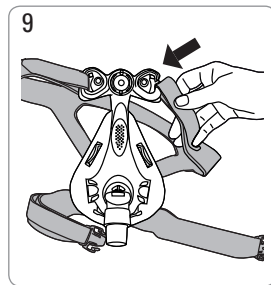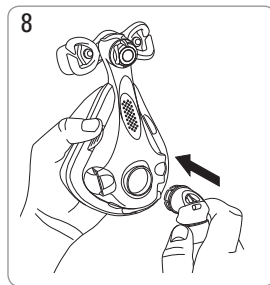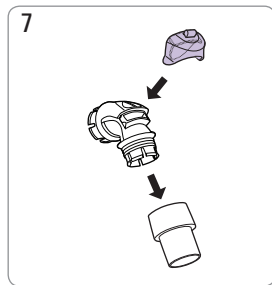Reassembly / Zusammenbau / Remontage / Riassemblaggio / Montaje / Montagem / Opnieuw monteren / Επανασυναρμολόγηση / Składanie / Összeszerelés / Повторная сборка / Opäťovné zloženie / Yeniden Montaj / Opětovné sestavení





Mirage Liberty™






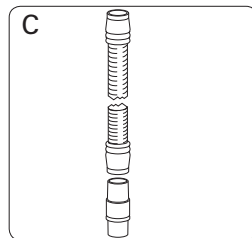
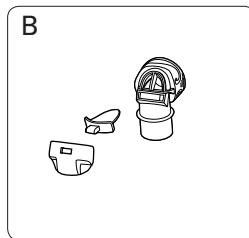
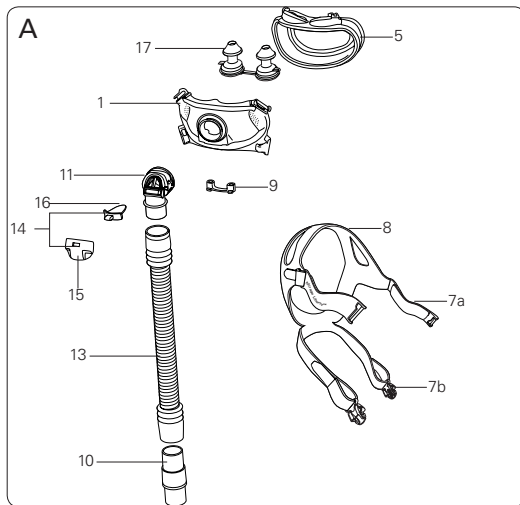
Mirage Liberty™

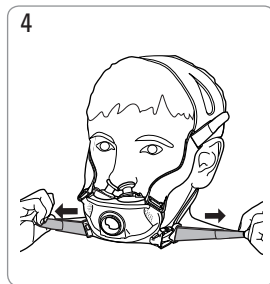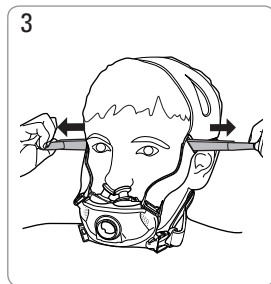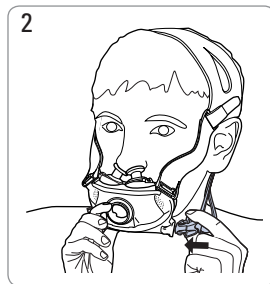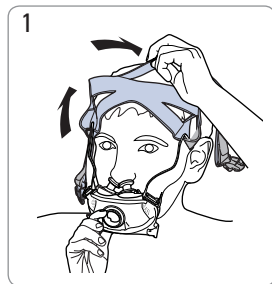Fitting / Anlegen / Mise en place / Applicazione / Colocación / Colocação / Opzetten / Τοποθέτηση της μάσκας / Zakładanie maski / A maszk felhelyezése / Установка маски / Nastavenie masky / Maskenin Takılması / Nasazení masky

Mirage Liberty™

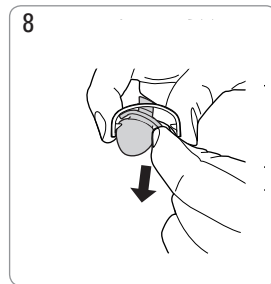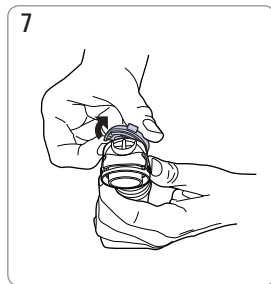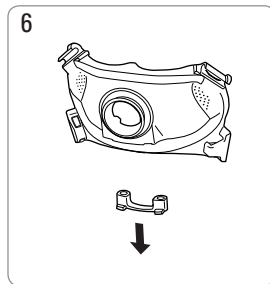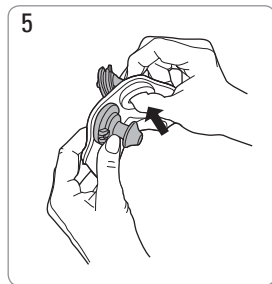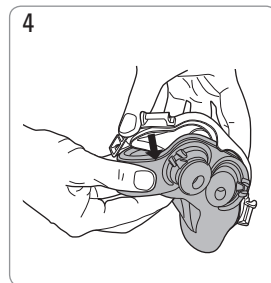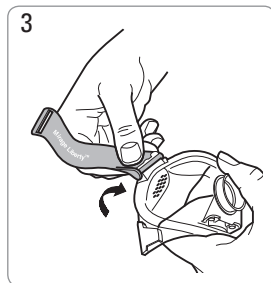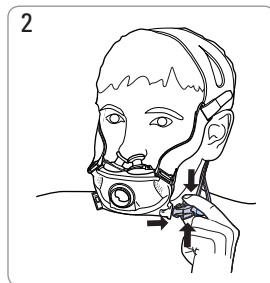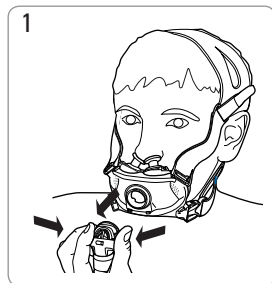
Disassembly / Auseinanderbau / Démontage / Smontaggio / Desmontaje / Desmontagem / Demontage / Αποσυναρμολόγηση /
Demontaž / Szétszerelés / Разборка / Rozobratie / Sökme / Demontáž


Mirage Liberty™

Reassembly / Zusammenbau / Remontage / Riassemblaggio / Montaje / Montagem / Opnieuw monteren / Επανασυναρμολόγηση / Składanie / Összeszerelés / Повторная сборка / Opätovné zloženie / Yeniden Montaj / Opětovné sestavení

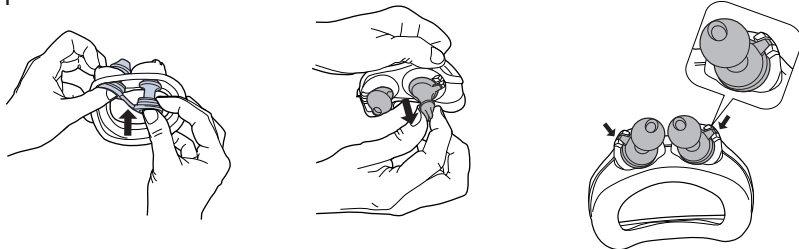
1


2

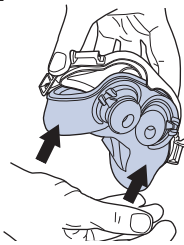
3

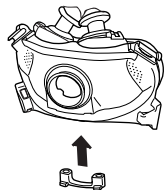4

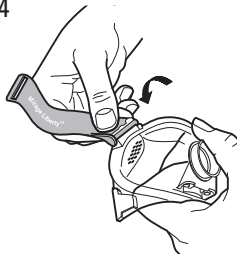5

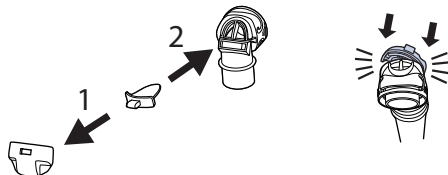English

VENTED FULL FACE MASK

Intended Use (Quattro Air/ Quattro Air for Her only)

The Quattro Air/Quattro Air for Her is a noninvasive accessory used for channeling airflow (with or without supplemental oxygen) to a patient from a positive airway pressure (PAP) device such as a continuous positive airway pressure (CPAP) or bilevel system.

The Quattro Air/Quattro Air for Her is:

- to be used by patients (weighing > 30 kg) for whom positive airway pressure therapy has been prescribed
- intended for single-patient reuse in the home environment and multipatient reuse in the hospital/institutional environment.

Intended Use (Quattro FX/ Quattro FX for Her/ Mirage Quattro/ Mirage Liberty)

The mask directs airflow noninvasively to a user from a positive airway pressure device such as a continuous positive airway (CPAP) or bilevel system.

The mask is intended for:

- adult users (>30 kg) for whom positive airway pressure has been prescribed
- single user re-use in the home environment and multipatient re-use in the hospital/institutional environment.

Using the Mask

- Refer to the Technical specifications section in this user guide for mask setting options.
- Follow the instructions provided by your physician or sleep therapist.
- (Quattro FX/Quattro FX for Her only) Prior to fitting your mask, remove the spring frame support from the rear of your mask and retain for future use.
- To fit your mask, follow the instructions in the Fitting section.
- Do not overtighten the headgear straps. If you feel air-leaks you may need to:

- refit or reposition your mask
- adjust your headgear
- check that your mask is assembled correctly
- contact your physician or sleep therapist to have your mask refitted.
- (Quattro Air/Quattro Air for Her only) Use a standard conical connector if pressure readings and/or supplemental oxygen are required.
- Mask-specific user guides are available on www.resmed.com on the **Products** page under **Service & Support**. They are also available from ResMed upon request.

This product does not contain natural rubber latex.

Device Settings

The mask setting options for ResMed devices are listed in the "Mask Compatibility Table" on www.resmed.com on the **Products** page under **Service & Support**.

Disassembly

To disassemble your mask, follow the instructions in the Disassembly section.

Reassembly

To assemble your mask, follow the instructions in the Reassembly section.

Cleaning the Mask at Home

The mask and headgear should only be gently handwashed.

Daily/After Each Use

- Disassemble the mask components according to the disassembly instructions.
- Thoroughly clean the separated mask components (excluding headgear), by gently rubbing in warm (approximately 30°C) water using mild soap.
- To optimise the mask seal, facial oils should be removed from the cushion after use.
- Use a soft bristle brush to clean the vent.
- Inspect each component and, if required, repeat washing until visually clean.
- Rinse all components well with drinking quality water and allow to air dry out of direct sunlight.
- When all components are dry, reassemble according to the reassembly instructions.

Weekly

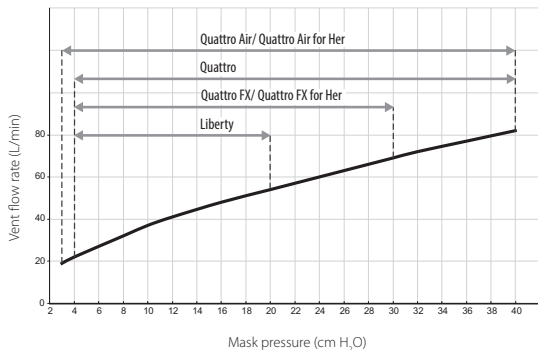
Handwash the headgear. It can be washed without being disassembled.

Reprocessing the mask between patients

The mask should be reprocessed when used between patients. Cleaning, disinfection and sterilisation instructions are available from the ResMed website, www.resmed.com/masks/sterilization. If you do not have internet access, please contact your ResMed representative.

Technical specifications

Pressure-flow curve – The mask contains passive venting to protect against rebreathing. As a result of manufacturing variations, the vent flow rate may vary.

	Quattro Air/ Quattro Air for Her	Quattro FX/ Quattro FX for Her	Quattro	Liberty
Pressure (cm H ₂ O)	Flow (L/Min)			
3	19	—	—	—
4	22	22	22	22
8	32	32	32	32
10	37	37	37	37
12	41	41	41	41
16	48	48	48	48
20	54	54	54	54
24	60	60	60	—
28	66	66	66	—
30	69	69	69	—
32	72	—	72	—
36	77	—	77	—
40	82	—	82	—

Deadspace: Empty volume of mask to swivel. The deadspace of the mask varies according to cushion size.

222 mL (Large)	243 mL (Large)	240 mL (Large)	151 mL (Large)*
-------------------	-------------------	-------------------	--------------------

* Empty volume of mask to elbow.

Therapy pressure (cm H₂O)

3 - 40	4 - 30	4 - 40	4 - 20
--------	--------	--------	--------

Resistance: Drop in pressure measured (nominal) (cm H₂O)

at 50 L/min	0.2	0.1	0.1	0.1
at 100 L/min	0.9	0.5	0.4	0.5

	Quattro Air/ Quattro Air for Her	Quattro FX/ Quattro FX for Her	Quattro	Liberty
Inspiratory and Expiratory Resistance (cm H ₂ O)				
Inspiration	0.6	0.7	0.8	0.7
Expiration	1.0	0.7	0.8	0.8
Anti asphyxia valve (AAV) open-to-atmosphere pressure (cm H ₂ O)				
	≤ 1.6	—	≤ 1.8	≤ 1.8
Anti asphyxia valve (AAV) closed-to-atmosphere pressure (cm H ₂ O)				
	≤ 1.8	—	≤ 1.8	≤ 1.8
Sound - DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level and the A-weighted sound pressure level of the mask, at a distance of 1M, with uncertainty of 3 dBA, are shown.				
Power Level	28 dBA	27 dBA	26 dBA	24 dBA
Pressure level at 1m	20 dBA	19 dBA	18 dBA	16 dBA
Gross Dimensions - Mask fully assembled - no headgear. Largest variant only shown.				
Height	172 mm	140 mm	206 mm	100 mm
Width	103 mm	176 mm	109 mm	110 mm
Depth	107 mm	144 mm	108 mm	100 mm
Mask setting options: Select A if available, otherwise select B .				
A	Full Face	Mir Full	Mir Full	Mir Full
B	Mir Full	Full Face	Full Face	Full Face
Environmental conditions – all masks:				
Operating temperature: +5°C to +40°C				
Operating humidity: 15% to 95% non-condensing				
Storage and transport temperature: –20°C to +60°C				
Storage and transport humidity: up to 95% non-condensing				

Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.
(Quattro FX/Quattro FX for Her only) To maintain optimum care of your mask, ensure the spring frame support is fitted when travelling.

Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

The following symbols may appear on the mask or packaging.

 Caution, consult accompanying documents;  Batch Code;  Catalogue Number;  European Authorised Representative;  Temperature limitation;  Humidity limitation;  Not made with natural rubber latex;  Manufacturer;  Keep away from rain;  This way up;  Fragile, handle with care.

Warnings

- The mask should only be used with CPAP or bilevel devices recommended by a physician or respiratory therapist.
- As with all masks, some rebreathing may occur at low CPAP pressures.
- The vent holes must be kept clear.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration varies depending on the pressure settings, user breathing pattern, mask, point of application and leak rate.
- If any visible deterioration of a mask component is apparent (cracking, tears, etc), the mask component should be discarded and replaced.

- Always follow cleaning instructions and use mild soap. Some cleaning products may damage the mask, its parts and their function, or leave harmful residual vapours that could be inhaled if not rinsed thoroughly.
- Your physician or sleep therapist is responsible for ensuring that the mask is compatible with the device. If used outside specification or if used with incompatible devices, the seal and comfort of the mask may not be effective and optimum therapy may not be achieved.
- Do not combine components from different mask systems.
- Discontinue using the mask and consult your physician or sleep therapist if you have ANY adverse reaction.
- Refer to your CPAP or bilevel device manual for details on its settings and operational information.
- This mask should be disinfected and/or sterilised when used between users.
- Remove all packaging before using the mask.
- The elbow and valve assembly have specific safety functions. They allow you to breathe fresh air if the airflow from your CPAP or bilevel device to your mask is impeded. The mask should not be worn if the valve is damaged as it will not be able to perform its safety function. The valve or elbow should be replaced if the valve is damaged, distorted or torn.
- The mask must be used under qualified supervision for users who are unable to remove the mask by themselves. The mask may not be suitable for those predisposed to aspiration.
- (Quattro FX/Quattro FX for Her only) Ensure the soft sleeves are fitted at all times during use.
- Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

Consumer Warranty

ResMed acknowledges all consumer rights granted under the EU Directive 1999/44/EC and the respective national laws within the EU for products sold within the European Union.

Clinical Information

The mask contains passive venting to protect against rebreathing.

Technical specifications, cleaning, disinfection and sterilisation instructions are available in the mask specific user guide on www.resmed.com on the **Products** page under **Service & Support**.

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Mirage, Mirage Quattro, Quattro and Mirage Liberty are trademarks of ResMed Ltd and Mirage is registered in U.S. Patent and Trademark Office. Velcro is a trademark of Velcro Industries B.V. © 2015 ResMed Ltd. 628090/3 2015-04
ResMed Ltd
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
EC REP ResMed (UK) Ltd 96 Jubilee Ave Milton Park Abingdon Oxfordshire OX14 4RW UK